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SILVERLON
PERFORMANCE. POWER. SPEED.

Silverlon®

Wound Contact Negative Pressure Dressings

Silverlon® Easy Ag™

Wound Contact Negative Pressure Dressings

EC
REP

Emergo Europe
Prinsessegracht 20
2514 AP The Hague
Netherlands
Tel. (+31) (0) 70.345.8570

Argentum Medical, LLC
2571 Kaneville Ct
Geneva, IL 60134 USA
Tel. (+1) 888.551.0188
Fax. (+1) 888.558.9923
www.silverlon.com

LC-CE-IFU-NPD-NPG-WCDNPS-03; Rev A

SILVERLON
PERFORMANCE. POWER. SPEED.

Silverlon® Negative Pressure Dressings
Silverlon® Easy Ag™ Negative Pressure Dressings

English

Device Description

Silverlon® Wound Contact Negative Pressure Dressing and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressing are sterile, single layer, non-adherent, antimicrobial barrier dressings, intended for topical for use in wound management and in negative pressure wound therapy (NPWT), to reduce risk of wound infection.

Available as:

Silverlon® NPD	Size	Silverlon® Easy Ag™	Size
NPD22	5 x 5 cm	WCD22-NPS	5 x 5 cm
NPD45	10 x 12 cm	WCD45-NPS	10 x 12 cm
NPD58	13 x 20 cm	WCD58-NPS	12 x 20 cm
NPD512	13 x 30 cm	WCD512-NPS	12 x 30 cm
NPD1212	30 x 30 cm	WCD66-NPS	15 x 15 cm
NPG – L	Glove	WCD88-NPS	20 x 20 cm
		WCD1212-NPS	30 x 30 cm

Silverlon® Wound Contact Negative Pressure Dressing and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressing contain silver to provide an ancillary, local antimicrobial effect to reduce the risk of wound infection by organisms sensitive to silver. Ionic silver is a broad spectrum antimicrobial, which has been shown to be effective against a range of pathogens associated with wound infection. Silverlon® dressings have been tested in vitro and found effective against microorganisms such as: Enterococcus faecalis, Staphylococcus aureus, Pseudomonas aeruginosa, Acinetobacter Baumanni, Enterobacter cloacae, Staphylococcus epidermidis and Klebsiella pneumoniae. Silverlon® Negative Pressure Dressing and Silverlon® Easy Ag™ Negative Pressure Dressing may be used with all types of negative pressure wound therapy porous foam/gauze dressings and equipment.

In vitro testing has shown Silverlon® dressings to be MRI compatible.

Indications

Silverlon® Wound Contact Negative Pressure Dressing and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressing are professional use wound dressings indicated for wound management and also use in conjunction with negative pressure wound therapy (NPWT), for local management of:

- 1st and 2nd degree burns
- Donor and graft site wounds
- Traumatic and surgical wounds
- Diabetic, pressure and venous ulcers
- Infected wounds*

*Silverlon® Wound Contact Negative Pressure Dressing and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressing may be used on those infected wound types listed above. Where used on infected wounds, the infection should be treated as per local clinical protocol.

Silverlon® Wound Contact Negative Pressure Dressings and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressing are indicated for a total duration of contact (i.e. where treatment involves consecutive application of individual dressings) of up to 30 days.

Warnings

- Do not use past expiration date on the product packaging.
- Do not use if pouch is damaged or open.
- Clinicians / Healthcare Professionals should be aware that there are very limited data on prolonged and repeated use of silver-containing dressings, particularly in children and neonates.
- Silverlon® Wound Contact Negative Pressure Dressing and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressings should not be used during pregnancy or lactation because of the lack of data concerning the effects of silver on reproduction.
- Silverlon® Wound Contact Negative Pressure Dressings and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressings are intended for single use only and should not be re-used. Re-use may adversely affect the dressing’s performance characteristics and also presents a risk of infection to the patient.
- Do not use petroleum-based ointments or creams under Silverlon® Wound Contact Negative Pressure Dressings and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressings.
- Do not moisten Silverlon® Wound Contact Negative Pressure Dressings and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressings with hydrogen peroxide or povidone iodine.
- Total duration of use of Silverlon® Wound Contact Negative Pressure Dressings and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressings (i.e. where treatment involves consecutive application of individual dressings) should not exceed 30 days.
- Silverlon® Wound Contact Negative Pressure Dressings and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressing

SILVERLON**Podtlakové obvazy Silverlon®**
Podtlakové obvazy Silverlon® Easy Ag™**Slovensky****SILVERLON****Silverlon® obloge z negativnim tlakom**
Silverlon® Easy Ag™ obloge z negativnim tlakom**Slovene****PERFORMANCE. POWER. SPEED.****Popis pomôcky**

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