

CMCcare Thumb Brace

by **BASKO** Healthcare®



- Careful design eliminates pressure in the web space and leaves the ulnar border of the hand free for function
- The adjustable embedded stay stabilizes the base of the first metacarpal on the trapezium to restore alignment and reduce pain
- Made of breathable, water resistant nylon, the 3pp Cozy Liner provides an extra layer of cushioning for additional comfort
- Replaceable strap is adjustable at both ends allowing the patient to determine the proper fit for stability and all-day comfort
- Durable thermoplastic polyurethane (TPU) is stain and abrasion resistant and can be easily adjusted with a heat gun for a custom fit
- Includes a 3pp® Cozy Liner, two closure tabs and a replacement strap; additional straps available

Indications

- ☒ CMC Thumb Arthritis
- ☒ Osteoarthritis
- ☒ Post-operative care
- ☒ CMC Thumb Pain

Features and Benefits

Removable 3pp® Cozy Liner cushions the thumb

Adjustable, replaceable strap for the perfect fit

Waterproof, durable and lightweight



Includes replacement strap and interchangeable closure tab & 3pp® Cozy Liner



Sizing Information

For the best fit and function you only need one measurement



Measure the length from your web space to your wrist crease

S	2" - 2 1/2" (5.1 - 6.4cm)
M	2 1/2" - 3" (6.4 - 7.6cm)
L	3" - 3 1/2" (7.6 - 8.9cm)

Healthcare Professionals

888-378-7763 M-F 8:30 - 5:00 EST
customerservice@3pointproducts.com

Patients & Consumers

Online Ordering
www.ohmyarthritis.com



ohmyarthritis.com

Order Numbers

Size	Right	Left
Small	P2014-R2	P2014-L2
Medium	P2014-R3	P2014-L3
Large	P2014-R4	P2014-L4

Application Instructions (Instructions included with each brace)



Hold the brace and press on the end of the stay to begin shaping it.



Slip the brace on and close the strap. Adjust the strap so the tab is centered on the hand.



With the brace on your hand, grasp the brace and "squeeze" it to conform it around the thumb.

Adjusting the CMCcare with a Heat Gun



Heat edges only until warm



Use the back of a spoon to carefully roll the edges

FAQs

What makes this orthosis a better option for treating my patient's CMC Thumb Arthritis?

1. **Comfort** - Smooth edges and carefully contoured design reduces pressure around the thumb and through the web space. Can be cushioned with a 3pp Cozy Liner or Gel Mate® silicone gel for added comfort.
2. **Fit** – Strap is removable and replaceable and is easily adjusted at both ends. A simple squeeze of the aluminum stay molds around the CMC joint. Edges can be flared using a heat gun for a custom fit.
3. **Function** - The functional design leaves the ulnar side of the hand free and stabilizes the CMC to allow unimpeded function for excellent patient compliance. The CMCcare is waterproof.

Cleaning Instructions

Clean regularly by hand washing with mild detergent or machine wash in a mesh laundry bag. Remove the strap and wash separately; lay flat to dry. Do not place in dryer. 100% Latex free

Suggested Billing Code

HCPCS code L3924 Use of this suggested code does not guarantee insurance reimbursement for this orthosis. Please consult with your medical billing specialist for assistance with billing questions. Code is current as of January 2020. PDAC letter available.