

**3pp®** ★Prima★ **Thumb Brace**

**NEW!**



Introducing a new benchmark in the design of a soft CMC and MCP support that's loaded with great features.

The **NEW 3pp Prima Thumb Brace** offers optimal support with a fully adjustable strapping system, memory-stretch material, and a removable dorsal stay. We are sure this will be your new go-to thumb brace to relieve CMC and MCP discomfort and instability.

**Features and Benefits**

Thumb strap allows for fully adjustable support

Removable stay adds CMC and MCP support

Easy-On Tab for easy one-handed application

Binding prevents fraying for long lasting wear

Memory-stretch fabric provides a contoured fit

Foam-lined material is breathable and water repellent



Contoured strap applies optimal support

Hook tabs secure the strap for consistent support

**Indications**

- CMC & MCP Arthritis
- Ligament Injuries
- Overuse Injuries

Suggested Billing Code  
HCPCS code L3924

**Sizing Information**



Small 5 1/2" to 7" (14 cm to 18 cm)  
Medium 7" to 8" (18 cm to 20 cm)  
Large 8" to 9 1/2" (20 cm to 24 cm)

Size	Left Hand Order#	Right Hand Order#
Small	P2022-L2	P2022-R2
Medium	P2022-L3	P2022-R3
Large	P2022-L4	P2022-R4

## Applying the 3pp Prima Thumb Brace



Slide thumb into the brace



Close the thumb strap with light tension



Hold the flap in place as you grasp and attach the Easy-On Tab



Grasp the strap or the edge of the flap and pull the brace closed



Stretch the strap lightly and bring the strap around the base of your thumb



Continue lightly stretching the strap around your thumb and over your web space



Bring the strap around the base of your thumb and attach the tab



The bendable plastic stay can be removed from the interior pocket as needed

### *What makes this orthosis a better option for treating my patient's thumb pain?*

**#1 Comfort** - Non-bulky, foam-lined material has natural memory so straps do not need to be pulled tight for a comfortable, customizable support. Binding eliminates fraying and breathable material helps dissipate heat.

**#2 Fit** - Straps attach anywhere on the loop material for a fully adjustable fit. Easy-On Tab makes applying the splint easy, and the wrap around thumb strap conforms around the CMC and MCP joints without restricting motion.

**#3 Function** - Carefully designed to conform in the palm so fingers are free for unimpeded function. The contoured thumb strap supports the CMC and MCP joints when pinching or gripping.

### To Order from 3-Point Products

Health Care Providers

888-378-7763


Mon-Fri 8:30am - 5:00pm EST

service@3pointproducts.com

### Online Ordering

Patients & Consumers

www.ohmyarthritis.com

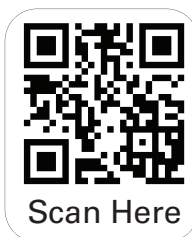

Scan Here