

1

Activate your kit online at color.com/covid/activate.




You will need your unique barcode, which you'll find on your collection tube.

If you did not reach the confirmation page or get an email / text confirmation, your kit is not activated!

WARNING: Your sample will not be tested if the kit is not activated.

2

Before collecting your sample, use hand sanitizer to clean your hands thoroughly.

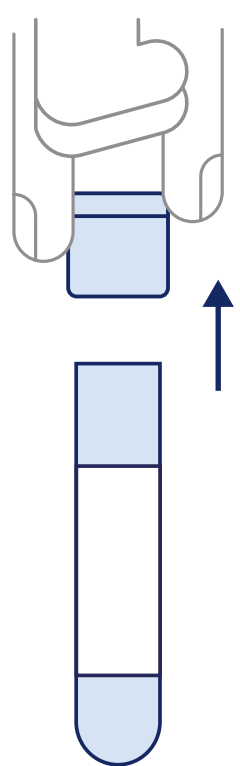

3

Remove the cap of the collection tube.

Keep the cap and collection tube nearby.

4

Open the package with the swab.

Starting from the handle end of the swab package, peel open the paper backing. Pull swab out of its packaging by the handle.


Note: Do not touch the soft tip with your hands or lay it down on any surface.

5

Rotate swab tip in first nostril, 4 times.

Insert the soft tip into one nostril, until it is no longer visible. Using medium pressure against the inside of your nostril the whole time, rub the swab in a large circular motion at least 4 times.

Note: Swab should not go further than half an inch.

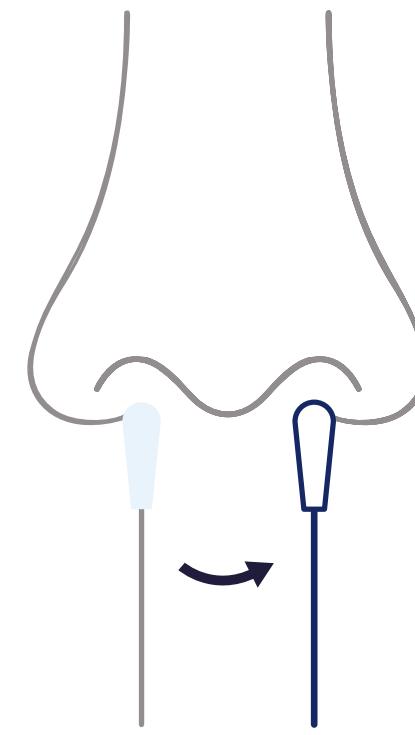

6

Rotate swab tip in other nostril, 4 times.

Repeat the previous step in the second nostril, using the same end of the swab.

Note: You will use the same swab for both nostrils.

7

Put the swab into the collection tube.

The soft tip of the swab that went into your nose should go into the tube first.



8

Push the cap of the collection tube closed.


You're almost done! Make sure the top is closed tightly.

Note: After closing the collection tube, use hand sanitizer to clean your hands thoroughly.

9

Put the collection tube into the biohazard bag.

Seal the bag closed.

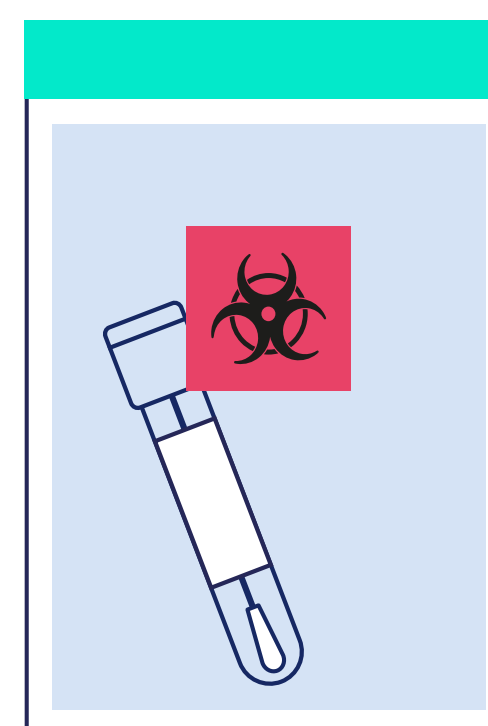

10

Bring the biohazard bag to your designated test collection point.

Refer to your program details if you're not sure where to deposit your sample.

