



iPACS® 2020 Release Notes

Release Notes: iPACS 2020

The Release: iPACS 2020

We're excited to announce the release of iPACS 2020!

iPACS 2020 has numerous features, enhancements and bug fixes outlined in the following pages. For more information, please contact info@invicro.com, Invicro Support at support@invicro.com, or Invicro Sales at sales@invicro.com. We look forward to your continued feedback and we thank everyone who has contributed through requests and reports.

Table of Contents

Features.....	3
Generic BPM Workflow.....	3
Submission 2020.....	3
Enhancements.....	3
Audit Log Enhancements	3
Additional Enhancements.....	4
Bug Fixes.....	4
Known Issues.....	4

Features

Generic BPM Workflow

- Upload BPMN workflow
- View BPMN workflow versions
- Associate BPMN process definition with iPACS context
- BPMN protocol configuration
- BPM Config - Download/Upload/Clone Version - include BPM datapoints
- Create Submission
- Form Configuration
- Email Notifications BPM Configuration
- Initial Review
- Query Pattern
- Workflow Tasks
- PDF Report
- Support CMMN Diagrams in the iPACS
- Implement Scan context
- Postable Process Variables
- Remove BPMN or CMMN Diagram from Active Context
- Workflow notifications sent out when:
 - task has no assignee
 - task is approaching due date
 - task has reached due date

Submission 2020

- Configure submission save location
- Preview individual Images in a Series during submission review
- Delete individual Images from a Series during submission review
- Delete non-DICOM files during submission review
- DICOM tags and sequences can now be removed during anonymization
- Submission Sites, Centers, and Contacts are now editable

Enhancements

Audit Log Enhancements

- Export log to password protected excel file
- Updated log export format from xls to xlsx, supporting up to 1.04 million log entries in an export
- Include time zone and seconds in log exports
- Added logging for the following actions:
 - Creating a DataPoint
 - Editing a DataPoint
 - Deleting a DataPoint
 - Uploading a BPM or CMM diagram
 - Starting an i2i Sync
 - i2i Sync completion
 - Saving a YAML form

- Locking a JSON form version
- Improved existing log messages include:
 - Before and after values for edits to user account information
 - Link to submission/process in log message for task completion
 - File counts in logs for completing and accepting submissions
 - Link to Patient in the import log for a Series
 - Link to Patient in the import log for an Image
- Log version of iPACS on system start/stop
- Added signature logs for the following actions, when the GxP module is enabled:
 - DICOM Browser moves and deletes
 - WebDisk moves and deletes

Additional Enhancements

- API: Admin:Forms - Admin level REST access to database forms
- JSON Forms: Implement locked JSON forms
- Security: Added support for Single Sign-On (SSO) using OAUTH and OpenID Connect

Bug Fixes

- [IPAC]-0010 - UI Errors when creating/editing project level data points
- [IPAC]-0021 - Sig Event link in audit trail leads to empty results and is confusing to users
- [IPAC]-0238 - Submissions: extract fails for zips with non-standard permissions
- [IPAC]-0563 - DICOM Dump fails for submission data
- [IPAC]-0588 - Cannot download the Merged Data after Static to Dynamic Pipeline run
- [IPAC]-0608 - Submissions: Edit patient information during IQC should not requires perms
- [IPAC]-0614 - Reports: cannot process PowerPoint .ppt reports
- [IPAC]-0615 - GxP: there is no 'Reason' field for GxP Deletes
- [IPAC]-1091 - irsync file from the image level does not include the UUID
- [IPAC]-1157 - Submissions: Chunked uploading large files doesn't display metadata
- [IPAC]-1355 - Fix console errors when loading protocol with no active config versions
- [IPAC]-1356 - Protocol config BPM tab, "Active Contexts" column fails to update
- [IPAC]-1359 - ApplyCoding: new UUIDs not generated consistently when "Patient" attributes are coded
- [IPAC]-1514 - Submissions: can't close "Edit Patient Information" window
- [IPAC]-1516 - Logging: DICOM Browser Move logs not associated with source project

Known Issues

- ipx-0335 - WD content indexing does not work with certain file types
- ipx-0346 - iPACS Sync "Quick Sync" doesn't work with ssh ports other than 22
- ipx-0361 - WD simple search for content returns fuzzy matches in addition to exact matches

- ipx-0369 - Storage usage stats on project overview page don't update on manual refresh
- ipx-0911 - The reason supplied when editing a locked DP is not recorded in the log message for the edit
- ipx-0965 - Filtering on the WD does not function if the filter string is an integer
- ipx-1004 - Form value from the Visual Read task in the General BPM is not cleared when the field is hidden
- ipx-1035 - Approaching due date notifications for tasks not sent as expected
- ipx-1038 - Error message not displayed when submitting invalid edits to Subject Information during IQC
- ipx-1060 - Submission data is not discarded when a submitter navigates away from the page
- ipx-1066 - Query response data is not discarded when a submitter navigates away from the page
Submitters cannot save a partial edit to the submission form during query response
- ipx-1076 - Deleting a single Image from a submission during IQC results in incorrect labels for the remaining Images
- ipx-1087 - Submitters using Internet Explorer cannot download submitted files from the submission creation page