

Taking Pooled Water Samples




Pooled or combined water samples are a cost-effective way to test the dental unit water quality coming out of your unit. Test every waterline in your unit (air-water syringe, high and low-speed handpieces, scalars, etc.) to most accurately test each unit. Your results will show the average bacteria count for all waterlines in that unit.

When to Test Your Dental Unit Waterlines:

- > Test your waterlines every month. Once passing results are consistent, test quarterly at a minimum.
- > Test your waterlines after completing any remedial action to ensure effectiveness
- > Test following any changes to your treatment protocol, equipment, source water, or plumbing.

QUICKPASS® VIAL

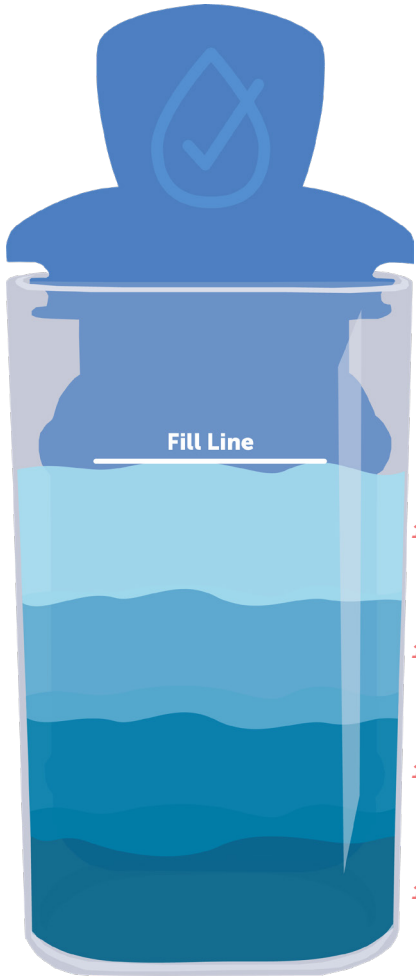

1. Determine the number of active waterlines on your dental unit that you will sample.

2. Based on that number, sample an equal amount of water from each device into the QuickPass® vial. For example, if you have 4 waterlines, fill the vial with 4 equal parts from each waterline (or 1/4 of the vial for each waterline). See figure for example.

NOTE: Do not combine a sample from source water with samples from dental units / devices. Always test source water separately.

3. Insert the QuickPass paddle and follow the next steps. On the Quickpass Waterline Testing Log (example below), under the "Device" section, checked "Combined" with the correct sampling date and location.

QUICKPASS TESTING LOG >

QuickPass® WATERLINE TESTING LOG

Device	Date of Results	Pass Or Fail	Safety Level	Corrective Action
<input type="checkbox"/> AW Syringe <input type="checkbox"/> Source Water <input checked="" type="checkbox"/> Scaler <input type="checkbox"/> Handpiece <input checked="" type="checkbox"/> Combined			● ● ● ●	
<input type="checkbox"/> AW Syringe <input type="checkbox"/> Source Water <input type="checkbox"/> Scaler <input type="checkbox"/> Handpiece <input type="checkbox"/> Combined			● ● ● ●	
<input type="checkbox"/> AW Syringe <input type="checkbox"/> Source Water <input type="checkbox"/> Scaler <input type="checkbox"/> Handpiece <input type="checkbox"/> Combined			● ● ● ●	
<input type="checkbox"/> AW Syringe <input type="checkbox"/> Source Water <input type="checkbox"/> Scaler <input type="checkbox"/> Handpiece <input type="checkbox"/> Combined			● ● ● ●	
<input type="checkbox"/> AW Syringe <input type="checkbox"/> Source Water <input type="checkbox"/> Scaler <input type="checkbox"/> Handpiece <input type="checkbox"/> Combined			● ● ● ●	
<input type="checkbox"/> AW Syringe <input type="checkbox"/> Source Water <input type="checkbox"/> Scaler <input type="checkbox"/> Handpiece <input type="checkbox"/> Combined			● ● ● ●	
<input type="checkbox"/> AW Syringe <input type="checkbox"/> Source Water <input type="checkbox"/> Scaler <input type="checkbox"/> Handpiece <input type="checkbox"/> Combined			● ● ● ●	
<input type="checkbox"/> AW Syringe <input type="checkbox"/> Source Water <input type="checkbox"/> Scaler <input type="checkbox"/> Handpiece <input type="checkbox"/> Combined			● ● ● ●	
<input type="checkbox"/> AW Syringe <input type="checkbox"/> Source Water <input type="checkbox"/> Scaler <input type="checkbox"/> Handpiece <input type="checkbox"/> Combined			● ● ● ●	



THEIR PROTECTION
YOUR ASSURANCE

888.843.3343 | support@pro edgedental.com

www.ProEdgeDental.com

©ProEdge Dental - v1

Questions?