

- 1) Power up the HealthGO Mini.



- 2) Wait until the end of initialization (at least 3 minutes). When initialization is complete, HealthGO Mini is in the following state:

*Note: Visual indicators shown in white may light in different colors and can be ignored for now.*



- 3) Press the button on the thermometer.

*Note: Before first use, insert the included batteries into the thermometer if they are not already inserted.*



- 4) Wait about 15 seconds before taking your temperature.

- 5) Take your measure with thermometer by gently placing thermometer at the entrance to the ear.

*Notes:*

- *Thermometer automatically initiates a communication with HealthGO Mini.*
- *Do not switch off the HealthGO Mini until the end of data transmission.*



*Note: The HealthGO Mini and the Thermometer must be in the same room for proper operation.*



QSG-EN-THERMO-IHEALTH-TS28B-REV-A-032020