

IBEX® LITE HR Firmware Update

[includes LITE HR, SuperLite HR, SuperLite/c HR]

If you would like to receive further instructions or assistance during business hours, we have a customer service representative waiting to assist you [866.365.6596].

Before you start...

1. Fully charge the battery.
2. Power the IBEX® **ON**
3. Insert the supplied compact flash card into the CF slot behind the door on the IBEX.
4. Connect the Ibex to your PC via the supplied USB cable. Your PC will show the IBEX as a drive under *My Computer*. The IBEX will display the message *Connected to PC*
5. Download the **.fw** upgrade file to the root of the CF card.
6. Disconnect the USB cable from the Ibex

Installing IBEX® LITE HR Series Firmware Update

1. Press the **SELECT** key—the system will display the Select menu.
2. Using the arrow keys highlight and select **SYSTEM**
3. Highlight and select **INSTALL**
4. Highlight and select **UPGRADE**
5. The system will display a list of all upgrade files that are on the CF card—highlight and select the desired upgrade file
6. Highlight and select **YES**
7. The system will apply the upgrade—when complete the system will display the message *Upgrade successful, press select to shutdown*
8. Press **SELECT**