

Protex™ Nebulizer Instructions

Product Number: SRNEB
Product Description: Nebulizer

1. Pour the test solution into the nebulizer as per the Protex™ applicator instructions found in the **Preparation** section of the Protex™ Qualitative Respiratory Fit Test User Instructions.
2. Remove the plugs from both the vent and the nebulizer exhaust port.
3. Ensure that the test solution is coming out of the exhaust port by compressing the squeeze bulb 4 to 5 times. (See Fig. 1)
4. If solution mist is not visible or seems very weak, carefully (so not to spill the solution) remove the top half of the nebulizer by unthreading it from the bottom half. (See Fig. 2)
5. Using your index finger lightly push down on the atomizer until it stops ensuring that it is pushed down all the way onto the bottom half of the nebulizer. (See Fig. 3)

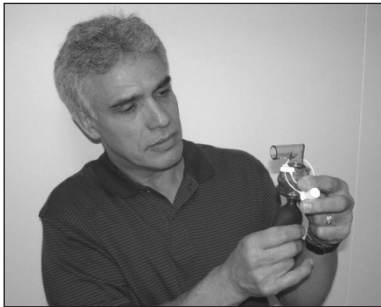


Fig. 1

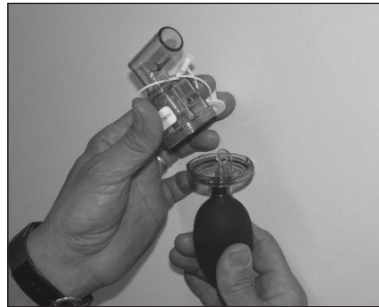


Fig. 2



Fig. 3

6. Thread the bottom half of the nebulizer back onto the top half of nebulizer.
7. Repeat step 3 to verify that the solution mist is coming out.
8. Perform the fit test as per the Fit Test Instruction Kit manual.

NOTE

If the solution mist is still weak or not visible, remove the atomizer and clean it using the supplied cleaning wire or replace it with one of the spare atomizers.

CAUTION

It is strongly recommended that the Fit Test Administrator wear a particulate respirator to protect them from prolonged exposure to the solution.

Shawmut Corporation
208 Manley Street
West Bridgewater, MA 02379
shawmutcorp.com/safety
508-588-3300

© 2021 Shawmut LLC. All Rights Reserved.
Lit #00071/0 04/21